Actimo® Home

DE ⚠

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Actimo® Home

UK 

Carefully read the entire user manual before assembling, adjusting or using the walking frame for the first time. Keep the instructions safely, so that they are available for later use.

Intended Purpose: A walking frame for indoor use for the assistance of people while walking who are restricted in their movements and mobility. Tray and netting for the carrying of utensils and items of daily use.

**Assembly / Instructions: Your specialist dealer or care assistant will be more than happy to help you with the assembly**

1. Before unpacking, check the packaging for damage. Should the contents be damaged, contact your specialist dealer immediately. The walking frame is delivered ready to use, is assembled by the specialist dealer or support staff and its function is checked.
2. Remove the walking frame and hold it firmly by the handle. Unfold the walking frame by pressing down on the upper cross running black strut. Check on the lower strut that the unfolding mechanism has been completely unfolded. Guide the two front bars on which the front wheels are found one after the other into the frame bar (see Illustration). Press in the small buttons situated on the sides. These must audibly snap into place and be visible at the side of the frame bar.

3. Guide the two bars on which the rear wheels with the brakes are found into the rear frame section (see Illustration). Press in the small buttons situated on the outside. These must audibly snap into place and be visible at the side of the frame bar. Ensure that the brake cables are not pinched or twisted!

4. In order to adjust the handle and the rest height, loosen the two wing nuts on the frame by turning them anti-clockwise (see Illustration). Remove the two screws one after the other and firmly hold the handle. Adjust the height (see Illustration) and push the two screws from the inside back through the holes in the frame again. Tighten them both by turning them clockwise.

5. To secure the netting, open the Velcro fasteners place them around the black frame and close them. Put the tray inside and carefully press it onto the frame.

6. To brake, pull up the lower lever on the handlebar. To lock the parking brake, press the brake lever down. To release the brake, pull the brake lever upward.
7. In order to fold up the walking frame, remove the tray and the netting. Firmly hold the walking frame by the handle with the brakes applied and pull the lower black strut upwards (see Illustration).

### Safety Advice

1. Never use a defective walking frame!
2. Do not use the walking frame if the brakes are defective and check them for their correct function before every use!
3. The walking frame is not a wheelchair nor a transport vehicle, but exclusively an aid to walking.
4. The tray and the netting must only be loaded with a maximum of 5 kg each.
5. Regularly check the walking frame for wear and tear and damage. Have damaged parts replaced by your specialist dealer.
6. Check the tightness of the fasteners and the correct height setting.
7. The Actimo® Home is intended for indoor use.  
Maximum load: 100 kg

### Cleaning

The Zimmer frame should be cleaned and disinfected at regular intervals (absolutely necessary when changing user). Use a mild soap solution without scouring additives. The disinfection can be carried out using a commercially available agent in accordance with the manufacturer's instructions, for example, Descosept AF. Do not use abrasive or caustic cleaning agents, high pressure or steam cleaners! Ensure that no dirt builds up on the tyres or the brakes. This can be removed with a soft brush and warm water.

### Maintenance

The Actimo® Home is maintenance-free.

Functional Faults: 

If you notice a reduction in the braking power or sluggishness in the wheels, contact your specialist dealer. Should you notice damage to or rust on the frame, inform your specialist dealer immediately and do not use a damaged walking frame!

### **Contraindications**

The use of Actimo® Home is unsuitable in the event of perceptual disorders, severe equilibrium disorders, reduces eyesight, loss of limbs on both arms and joint damage on the arms.

### **Scope of Supply**

Actimo® Home  
Tray  
Net  
Instructions for use

### **Technical Data / Dimensions and Weight:**

Weight: 6,2 kg; Material: Aluminum - Frame (corrosion protection – anodic oxidation), EVA (ethylene vinyl acetate) handle  
Wheels: PVC 6“  
Maximum load: 100kg  
Width: 58 cm  
Depth: 60 cm  
Adjustable height: 76 – 86 cm  
Dimensions of the folded walking frame: approx. 52 x 51 x 21 cm  
External dimensions (packaging): approx. 54 x 53 x 23 cm

### **Guarantee**

The guarantee period is 24 months and relates to all product defects which can be verifiably shown to be attributable to manufacturing or material defects.

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